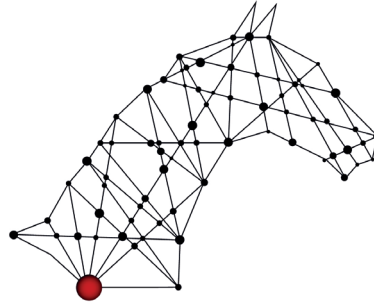
ARIONE
HORSE DATA SCIENCE

INSTALLATION GUIDELINES

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Harness System

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EN ENGLISH

Harness System 1

How to install EQUIMETRE? 1

How to take the EQUIMETRE off? 4

How to synchronise your data? 5

FR FRANÇAIS

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Comment installer EQUIMETRE ? 6

Comment enlever EQUIMETRE ? 9

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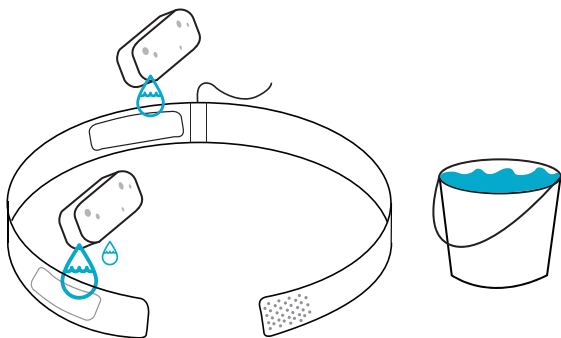
EN HARNESS SYSTEM

How to install EQUIMETRE?

The installation of your EQUIMETRE begins before you start saddling.

1 WET THE 2 ELECTRODES


Warning : Do not drown the wire.

Wet both foams.


The surcingle should never be used without the foams.

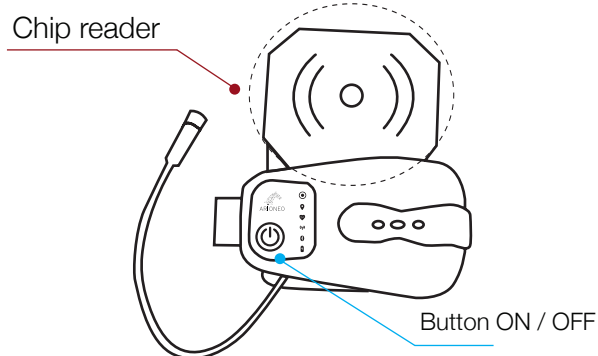3 TURN ON YOUR MONITOR


Turn on the EQUIMETRE by pressing the button once. The first white LED should blink.

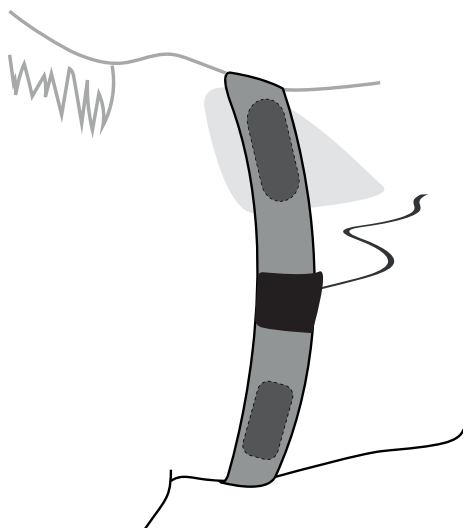2 INSTALL THE SURCINGLE

Install the surcingle so that the **upper electrode is placed in the withers hollow, on the left**, in direct contact with the horse's skin.

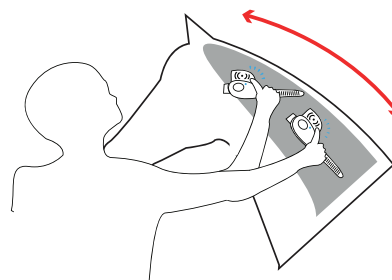4 LOOK FOR THE HORSE'S VETERINARY CHIP


Look for your horse's veterinary chip by scanning the neck from withers to ears along the mane **SLOWLY**, until you hear a bip.

OR


Press the button 3 times to launch a training manually. You'll have to assign the horse's identity at the end of the training.

A bip sound indicates the start of the training. The first white LED should stop blinking.

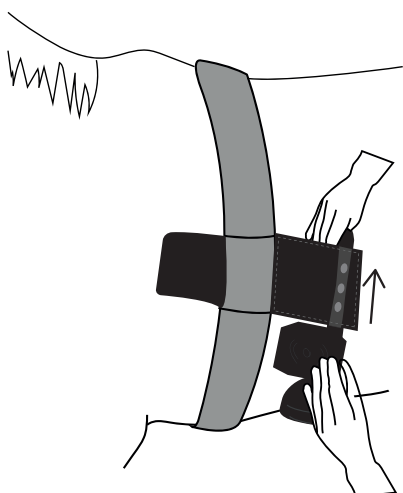
5 INSTALL THE SENSOR IN THE SURCINGLE POCKET

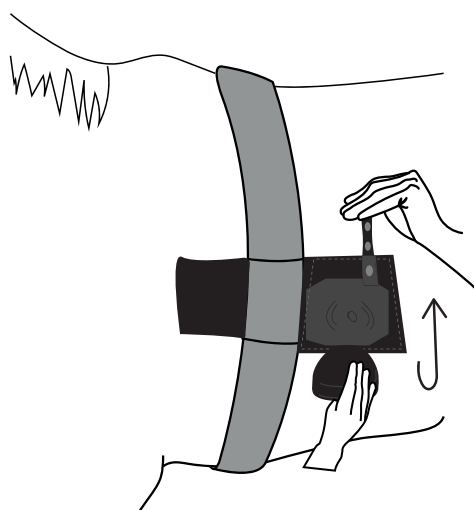Insert the chip reader into the big hole of the surcingle pocket using both hands.



Get the strap through the small hole.

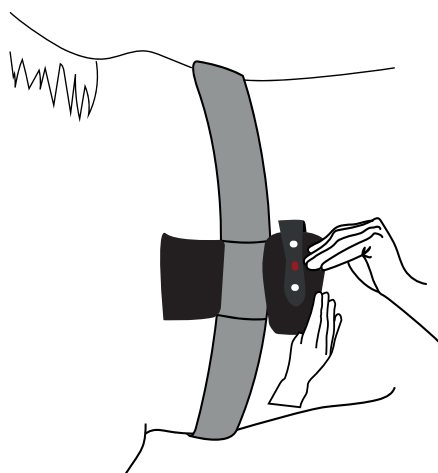
Secure the sensor by fastening the strap. The sensor is outside the pocket, and the chip reader is inside.

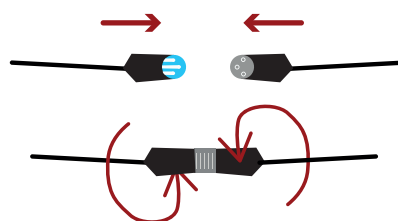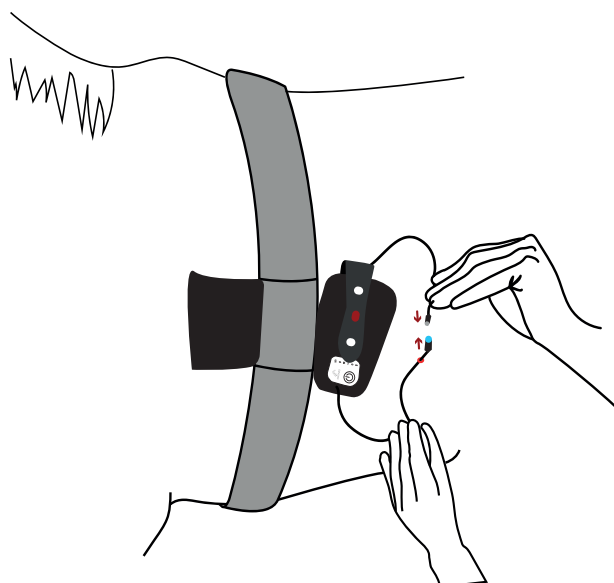6 CONNECT THE SURCINGLE AND SENSOR WIRES


Connect the two wires by aligning the 3 pins, and screw in opposite directions.

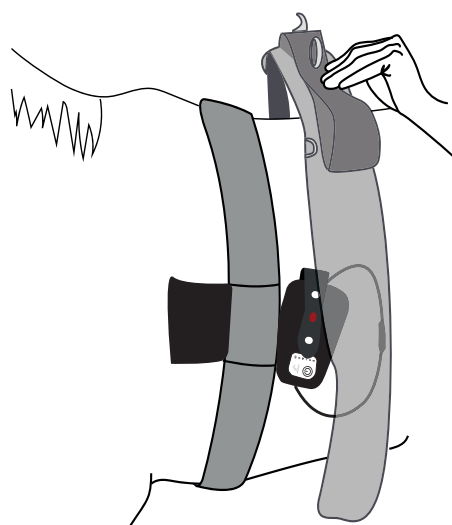7 PUT THE HARNESS ABOVE THE SURCINGLE


Install the harness on your horse as you usually do.

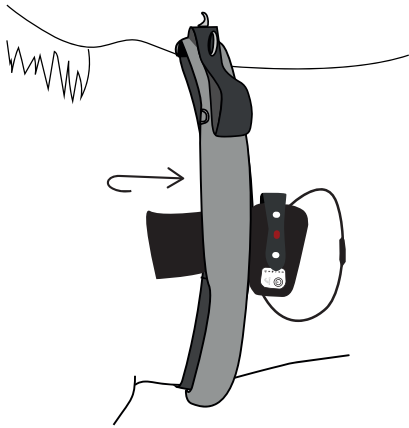8 ATTACH THE VELCRO TO THE HARNESS

Open the pocket and attach the velcro back to secure the surcingle around the harness.

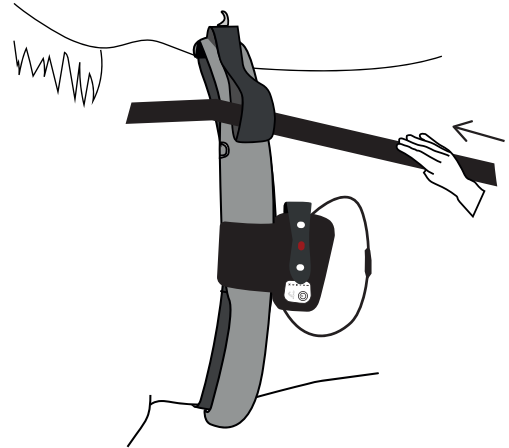9 INSTALL YOUR SULKY

You can then install your sulky as you usually do.

10 CHECK IF YOU'RE READY TO GO

The first white LED must be turned on and white.

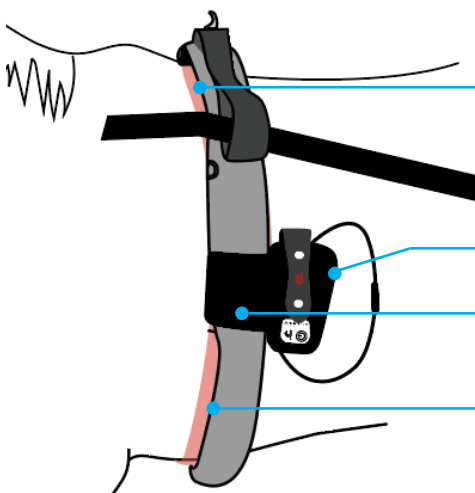
The second green LED shows GPS and flashes until a signal is detected. When you are inside, it may flicker; when you are outdoors, it will stabilize.

The third red LED shows the heart rate. Once installed, the sensor should blink at your horse's heart rate.

11 YOU'RE GOOD TO GO IF

The surcingle is installed under the harness, against the horse's skin.

The sensor is in the surcingle pocket and the cables are connected.

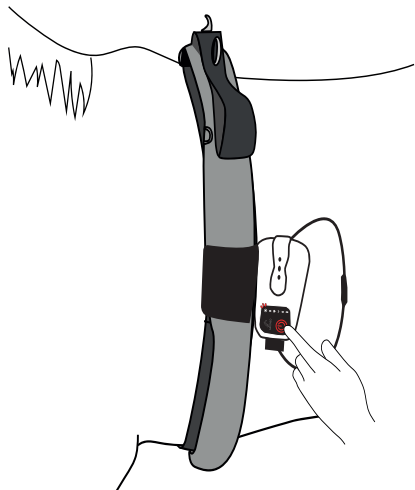
The velcro is attached around the harness.

The surcingle is tightened to ensure good conductivity of the heart signal.

How to take the EQUIMETRE off?


At the end of your training, it is important to remove your EQUIMETRE by following the instructions below.

1 TURN THE EQUIMETRE OFF


Turn the EQUIMETRE off by pressing the button during 3 secondes. All LEDs should turn off.

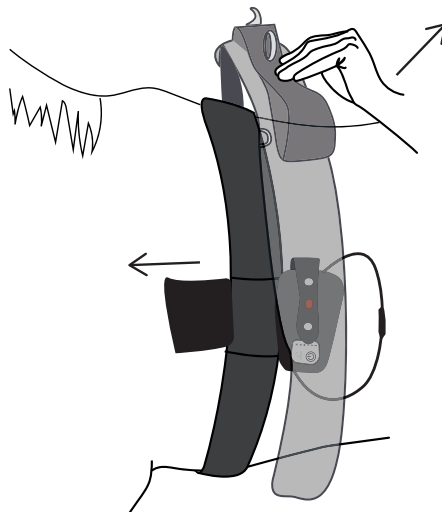2 TAKE THE HARNESS OFF


Open the velcro and take the harness off.

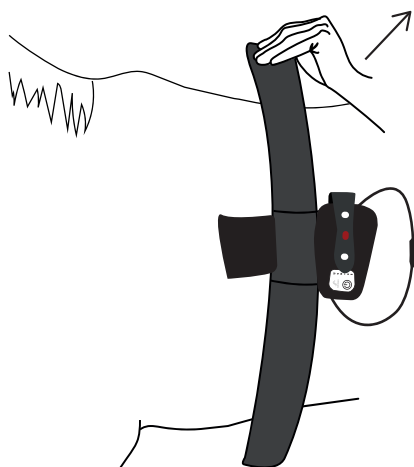3 TAKE THE SURCINGLE OFF


Take the surcingle off your horse, but you can let the EQUIMETRE in the pocket.

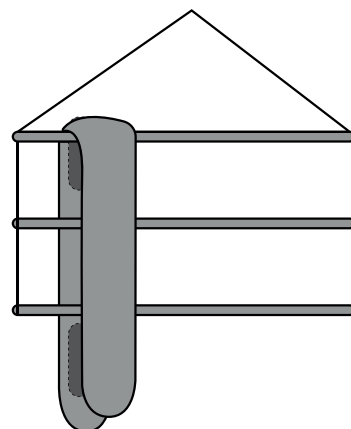4 HANG THE SURCINGLE BEFORE THE NEXT USE

Warning: Hang your surcingle to prevent the cables inside from breaking, and to allow it to dry properly.

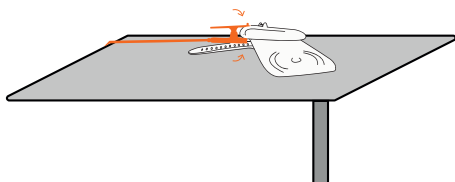5 CHARGE THE EQUIMETRE

Warning : When not in use, be sure to always plug EQUIMETRE and check if the sensor is turned off. The charging orange LED should be on.

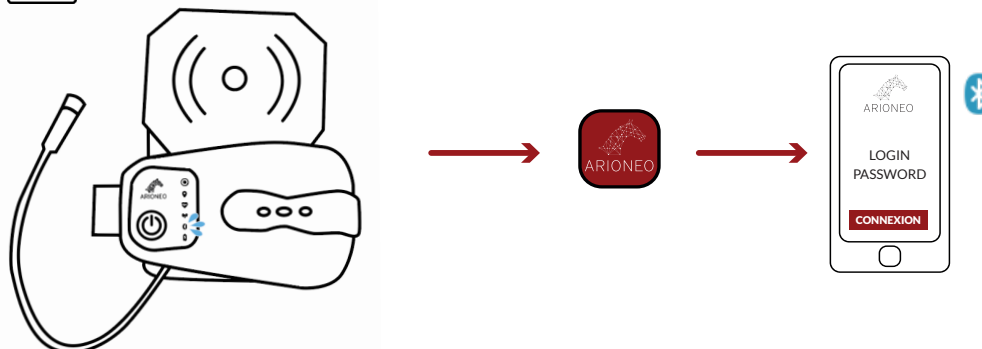
How to synchronise your data?

At the end of your training, access your data with your phone.

1 LOG IN TO THE EQUIMETRE APP


TURNED-OFF & NEARBY

2 CLICK ON THE «SYNCHRONISE» BUTTON

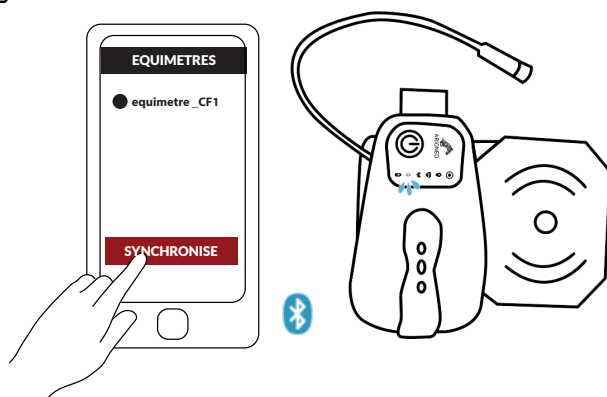
TURNED-OFF & NEARBY

The download gauge reaches 100%.
EQUIMETRE must be turned off.

Your phone must be close to the sensor. The
bluetooth LED should start blinking.

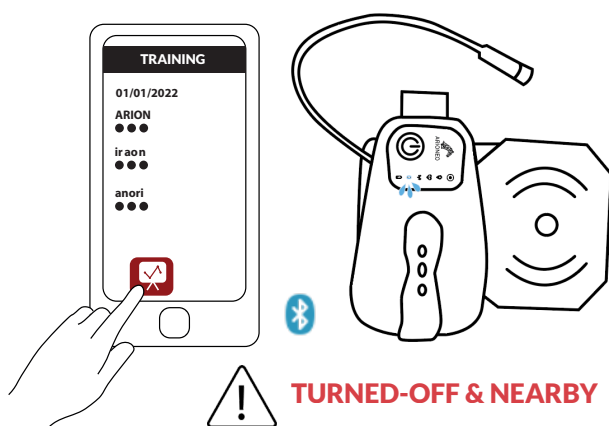3 IN THE «TRAINING» TAB YOU WILL FIND THE LIST OF DOWNLOADED TRAINING SESSIONS

TURNED-OFF & NEARBY

Trainings are automatically uploaded to the Cloud so
you can find them from any device on Equimetre.com.

Click on the training to view it directly from the app.

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